

STRYKER BATTERY CHARGER INSTALATION

September 26,
2017
Page 1 of 1

AMBULANCE TYPE: **CRESTLINE**

UNITS AFFECTED: **ALL UNITS EQUIPPED WITH THE STRYKER POWERPRO
STRETCHER**


ACTION: **INSTALLATION OF THE STRYKER POWERPRO BATTERY CHARGER**

LABOUR TIME: **.3 HRS BILL TO: BCAS**

SERVICE PROCEDURE:

1. Remove the storage bins to provide space to mount the Charger on the left of the lower shelf located in the cabinet above the suction unit.
2. Position the mounting base $\frac{1}{2}$ inch from the side wall and flush to the front cabinet door track.
3. Using the charger base guide pins as a template, drill two $\frac{1}{8}$ th holes into the lower shelf.
4. Squeeze two $\frac{1}{4}$ th beads of silicone onto the base and install on the shelf with the supplied self-tapping screws.
5. Insert the power cord and mount the charger onto the base. Stick the double sided tape to the transformer and attach to the wall behind the charger. Install split loom onto the power cord and route beside shelf. Plug in to 110V power and secure the excess cables with zip ties.



| | | |
|---|---|---|
|  <p>BC Ambulance Service BC Emergency Health Services</p> | <p>STRYKER BATTERY CHARGER INSTALATION</p> | <p>September 26, 2017 Page 2 of 1</p> |
|---|---|---|