



AMBULANCE TYPE:

Crestline

UNITS AFFECTED:

63003 – 63005 and Remounted 63000 series

ISSUE: Suction unit upgrade to disposable canisters.


ACTION: Replace existing suction unit with supplied upgrade unit.

Labour time: .8 hr Please use labour code **GMOSU** on invoice.

SPECIAL TOOL: 1 1/8" Hole saw

SERVICE PROCEDURE: *Read through the instructions before starting*

- 1) Remove existing suction unit and discard.

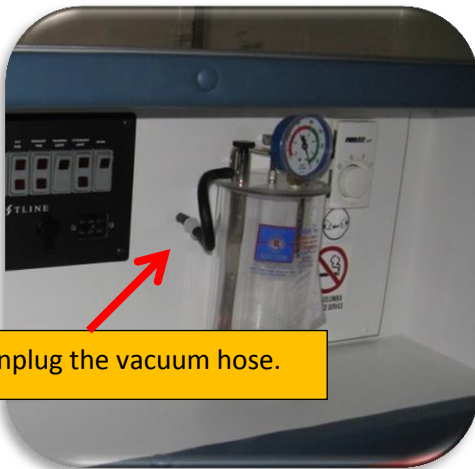

Unplug the vacuum hose.

Remove canister by unscrewing the thumb screw.



Remove the 4 screws.



AMBULANCE TYPE:

Crestline

UNITS AFFECTED:



63003 – 63005 and Remounted 63000 series

2) Install new suction unit.



1) Remove the plastic fitting from the vacuum hose.

2) In the Electrical compartment, remove the access panel to expose the vacuum hose – pull the hose out through the hole.

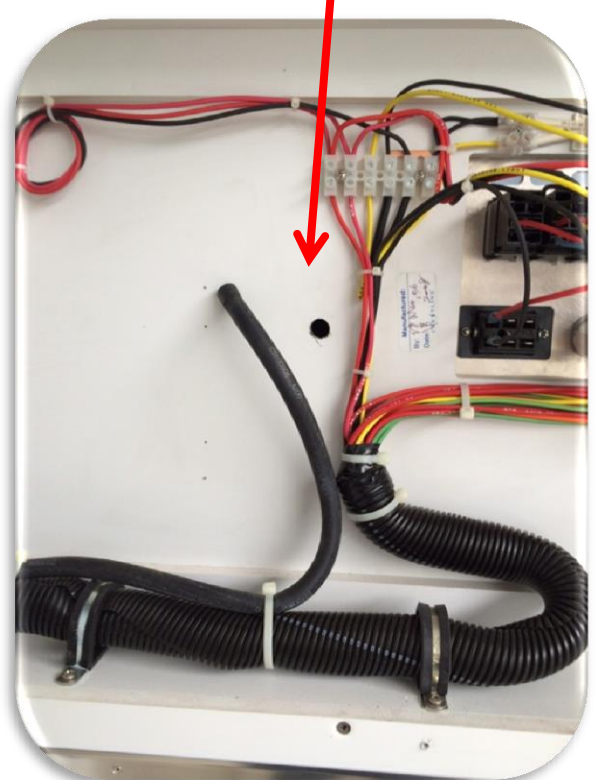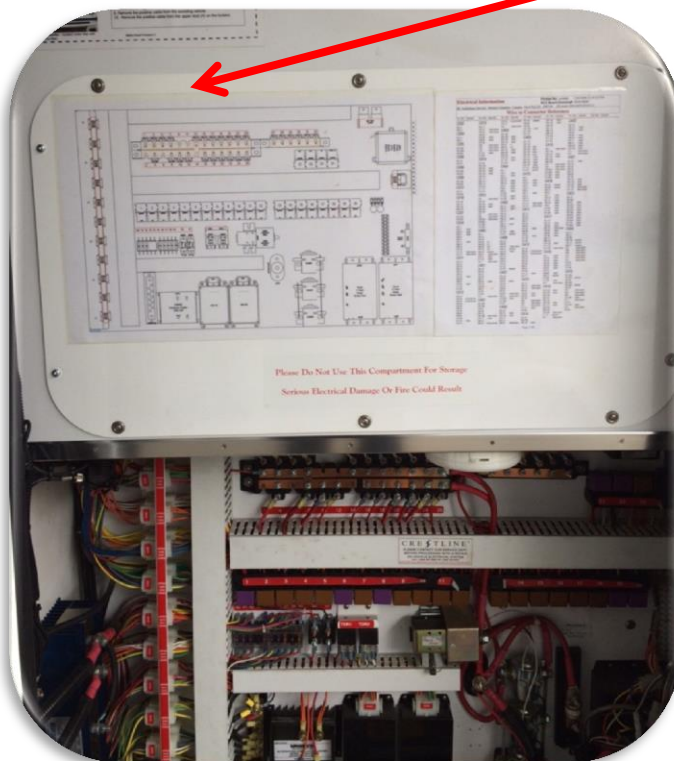



AMBULANCE TYPE:

Crestline

UNITS AFFECTED:

63003 – 63005 and Remounted 63000 series

On the interior wall, center the vacuum face plate over the hole where the vacuum hose came through the wall. Mark the retaining screw holes. Be sure the face plate is square on the panel.



Use a hole saw to cut a 1 1/8" hole for vacuum hose. Drill appropriate size pilot holes for the retaining screws.





AMBULANCE TYPE:

Crestline


UNITS AFFECTED:

63003 – 63005 and Remounted 63000 series

Assemble the vacuum hose fittings and face plate.



Install the face plate to the wall.
Attach the vacuum hose in the electrical compartment and re-install the cover panel.

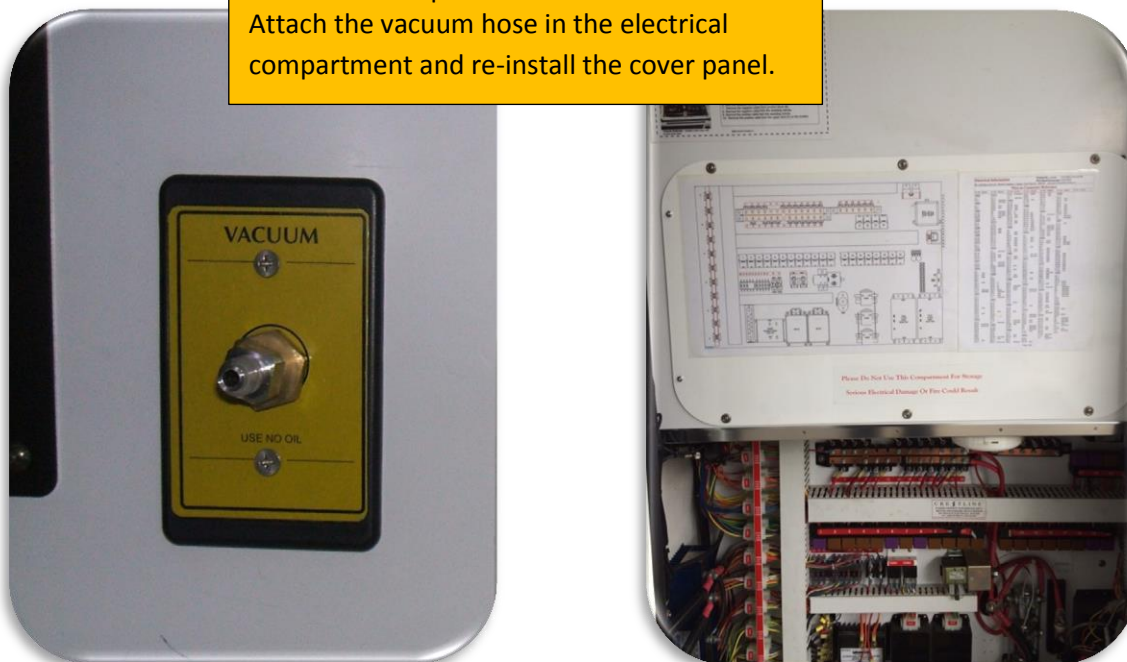



BC Ambulance Service
BC Emergency Health Services

Rico Suction Unit RS-4X Upgrade


January 16, 2017
Page 5 of 6

AMBULANCE TYPE:

Crestline

UNITS AFFECTED:

63003 – 63005 and Remounted 63000 series

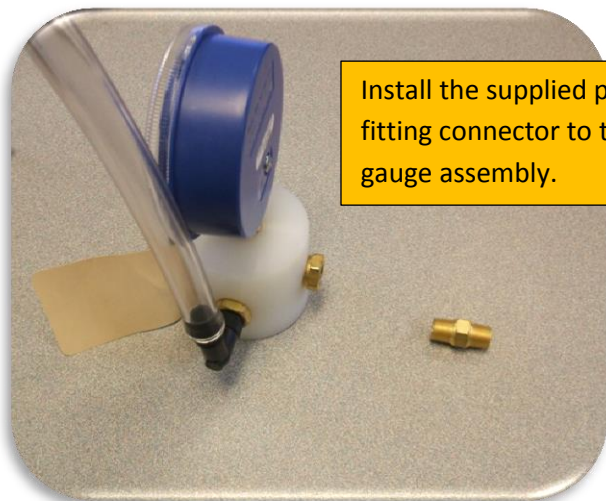

Install the supplied pipe fitting connector to the gauge assembly.



Thread the gauge assembly into the wall socket.



Install supplied wall mount plate – locate the plate just above the bottom hole where the previous suction unit mounted – use the wall mount plate as a template to drill the screw holes – install the supplied screws.



Install the canister mount ring.



BC Ambulance Service
BC Emergency Health Services

Rico Suction Unit RS-4X Upgrade

January 16, 2017
Page 6 of 6

AMBULANCE TYPE:

Crestline

UNITS AFFECTED:

63003 – 63005 and Remounted 63000 series



Drill a 1/8" pilot hole through the mount ring and into the wall. Drill a 7/32" hole through the mount ring and plate – do not drill into the wall. Install the retaining screw as shown. This will prevent the ring from being removed with the canister.

Install the new canister into the mounting ring.



Connect all vacuum hoses

