

### AMBULANCE TYPE: MX151D Units: - 63060 to 63079, 63100 to 63149, 63200 to 63249

## ISSUE

This bulletin is to inform owners of Demers Ambulances (unit numbers listed above) that a modification of the air conditioning unit is required. It has been brought to our attention that some AC unit may spit water when in use.

## ACTION

Modification of the AC unit is required during the next scheduled 'A' service safety inspection. Please perform the modification as per the following link.

Instruction should be follow as per the video once opening the link: <u>https://youtu.be/ei\_ZcuN6TCA</u>

During the modification please check the AC drain tubes located under the vehicle behind the driver door and in front of the electrical compartment, under the vehicle and remove the cap on the drain (if present).



# SERVICE PROCEDURE

Modification time: 0.5 hour / per vehicle.

Qty.	Description	Demers Part #
1/2 per Unit	FLEXIBLE URETHANE FOAM SEM 39357	E206028
1	UNIT MCC13-2563 SIDE COVER	D027144

#### Please note:

With our new paperless warranty system no form is required to be filled out as these recalls are <u>pre-approved</u> for the time allotted, please fax your invoice to Demers Customer Assistance Centre service department at: 1-306-373-4144.

\*Please be sure to indicate the service bulletin number on the invoice.

Should you require additional time or have questions please call our Demers Customer Service Centre toll free at **1-855-373-9789**.