

 BC Ambulance Service BC Emergency Health Services	Stair Chair Bracket Mod – Power Stair Chair Trial	February 13, 2018 Page 1 of 1
AMBULANCE TYPE:	Demers	
UNITS AFFECTED:	63414, 63450, 63302, 63455, 63055, 62803	

ISSUE:

The current Stair Chair Bracket needs to be modified to accommodate the Ferno Power Stair Chair.

ACTION:

Install the supplied Spacer Kit pieces behind the current Stair Chair Bracket.

LABOUR TIME: .3 HRS **Bill to:** BC Ambulance

SERVICE PROCEDURE:

1. Remove the current Stair Chair Bracket, shown below. (Discard hardware)



2. Install current Stair Chair Bracket with the supplied spacer kit and hardware, shown below.(apply one drop of blue thread Loctite to each supplied screw)

