



**British Columbia
Ambulance Service**
Emergency and Health Services Commission

Memorandum

Fleet Operations
PO Box 9615 Stn Prov Govt
3300 Douglas St
Victoria BC V8W 9P1

ATTENTION: All BCAS SERVICE CENTERS

Re: Stair Chair Restraint Belt Anchor Bolt Upgrade - Demers

To prevent loosening of restraint belt for stair chairs, the mounting hardware for the restraint belt must be updated. BCAS Fleet Operations request the following procedure to be performed to all Demers built vehicles 62800 - 92981. **Crestline built vehicles do not apply.**

Procedure:

Remove the screw holding the stair chair belt from right and left side of side patient door and discard screws.

Drill 25/64" holes into door at existing screw hole locations.

Install the supplied Nutsert and reinstall the belt using supplied hardware with lock tight.

Belt anchors must be installed pointing toward outside edges of the door. (See picture)



Hardware kits have been sent to affected service centers.

The labour time allowed for this modification is **.6 hrs.**

Please identify this repair as **GMODCBR** on your invoice.

GMODCBR – Stair Chair Belt Hardware Upgrade

For any questions or concerns please call Fleet Operations at **1-877-652-7465**.

Please completely read modification procedure before proceeding with repair:

British Columbia Ambulance Service

Professionalism - Accountability - Responsiveness - Teamwork - Wellness