 BC Ambulance Service BC Emergency Health Services	Rico Suction Unit RS-4X Upgrade	July 17, 2015 Page 1 of 4
AMBULANCE TYPE:	Demers	
UNITS AFFECTED:	62800 - 62899 / 62911 – 62981 / 63000 – 63002 / 63030 - 63164	

ISSUE: Unit upgrade to disposable canisters

ACTION: Replace existing suction unit with supplied upgrade unit

Labour time: .3 hr Please use labour code **GMOSU** on invoice


SERVICE PROCEDURE:

- 1) Remove existing suction unit
 - Unplug suction hose from wall socket



- Turn thumb screw counter clockwise to remove canister





 BC Ambulance Service BC Emergency Health Services	Rico Suction Unit RS-4X Upgrade	July 17, 2015 Page 2 of 4
AMBULANCE TYPE:	Demers	
UNITS AFFECTED:	62800 - 62899 / 62911 – 62981 / 63000 – 63002 / 63030 - 63164	

- Remove canister wall mount bracket



2) Install supplied suction unit

- Remove wall fitting from suction hose on removed canister and thread into new gauge

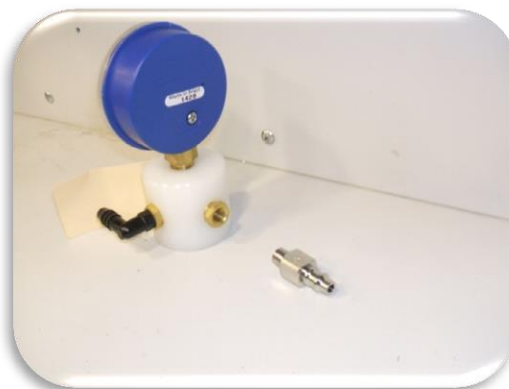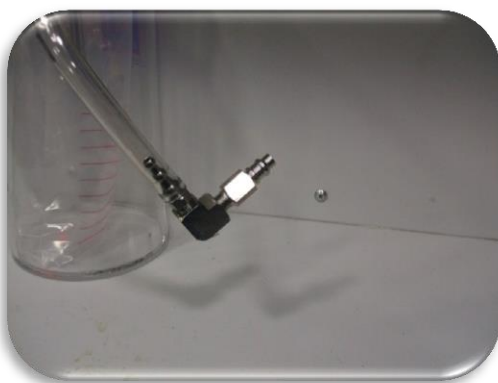




AMBULANCE TYPE:

Demers


UNITS AFFECTED:

62800 - 62899 / 62911 – 62981 / 63000 – 63002 / 63030 - 63164


- Secure mount plate to wall

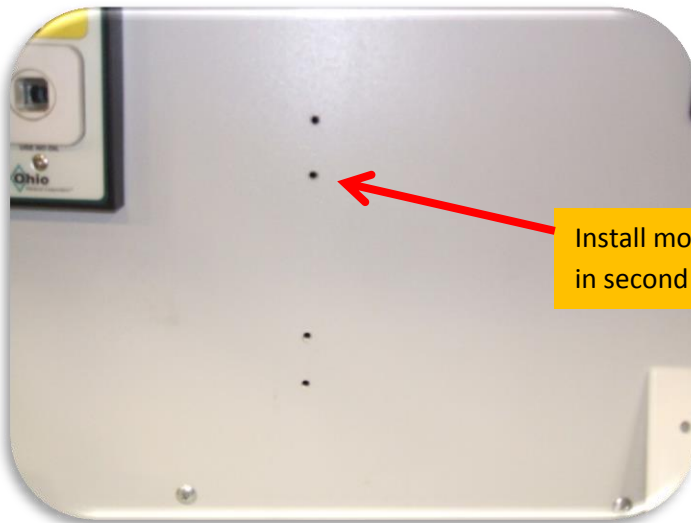

Install mount plate screw
in second hole down

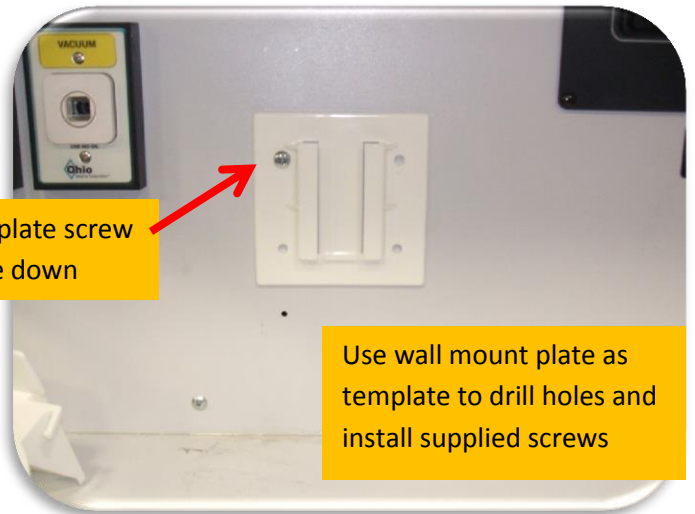

Use wall mount plate as
template to drill holes and
install supplied screws

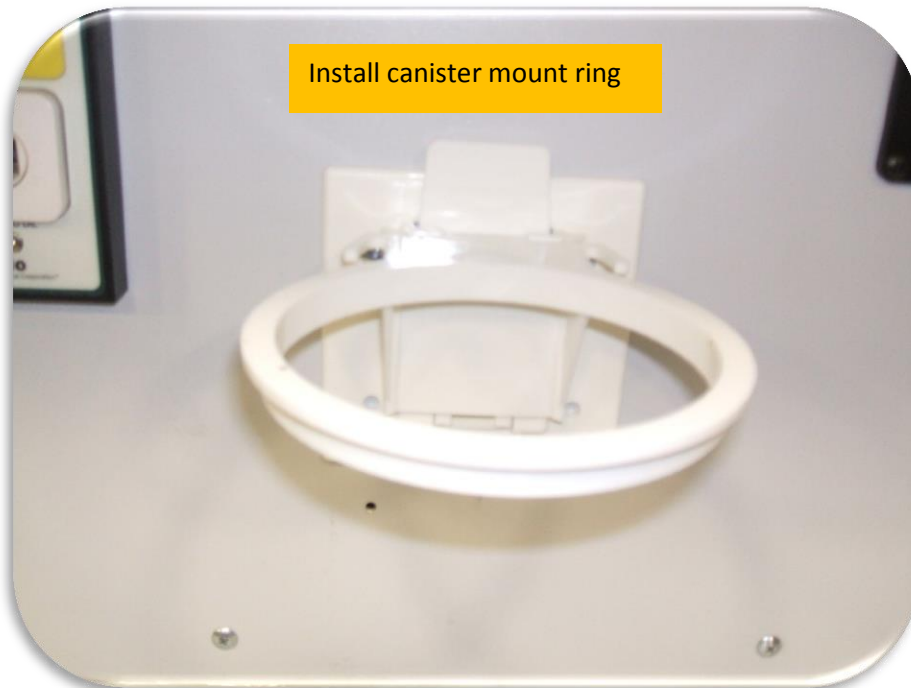

Install canister mount ring



AMBULANCE TYPE:

Demers

UNITS AFFECTED:

62800 - 62899 / 62911 – 62981 / 63000 – 63002 / 63030 - 63164

- Plug suction gauge unit into wall socket (fitting should “click” when properly installed)



- Install canister and connect vacuum hoses

