

July 17, 2015 Page **1** of **4**

AMBULANCE TYPE:

Demers

UNITS AFFECTED:

62800 - 62899 / 62911 - 62981 / 63000 - 63002 / 63030 - 63164

ISSUE: Unit upgrade to disposable canisters

ACTION: Replace existing suction unit with supplied upgrade unit

Labour time: .3 hr Please use labour code GMOSU on invoice

SERVICE PROCDURE:

1) Remove existing suction unit

Unplug suction hose from wall socket





Turn thumb screw counter clockwise to remove canister



1 877 652-7465

Fax: 250 952-0275



July 17, 2015 Page **2** of **4**

AMBULANCE TYPE:

Demers

UNITS AFFECTED:

62800 - 62899 / 62911 - 62981 / 63000 - 63002 / 63030 - 63164


Remove canister wall mount bracket



2) Install supplied suction unit

Remove wall fitting from suction hose on removed canister and thread into new gauge



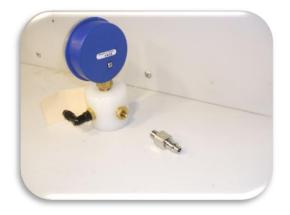

1 877 652-7465 Fax: 250 952-0275



July 17, 2015 Page 3 of 4

AMBULANCE TYPE:

Demers

UNITS AFFECTED:

62800 - 62899 / 62911 - 62981 / 63000 - 63002 / 63030 - 63164

Secure mount plate to wall





1 877 652-7465

Block C, 2261 Keating X Road

Fax: 250 952-0275



July 17, 2015 Page **4** of **4**

AMBULANCE TYPE:

Demers

UNITS AFFECTED:

62800 - 62899 / 62911 - 62981 / 63000 - 63002 / 63030 - 63164

Plug suction gauge unit into wall socket (fitting should "click" when properly installed)



Install canister and connect vacuum hoses



1 877 652-7465 Fax: 250 952-0275