

STRYKER BATTERY CHARGER INSTALLATION

September 26,
2017
Page 1 OF 1

AMBULANCE TYPE: DEMERS 63000 AND UP

UNITS AFFECTED: ALL UNITS EQUIPPED WITH THE STRYKER POWERPRO
STRETCHER

ACTION: INSTALLATION OF THE STRYKER POWERPRO BATTERY CHARGER

LABOUR TIME: .3 HRS **BILL TO:** BCAS

SERVICE PROCEDURE:

1. Charger is to be mounted in the Monitor compartment on the right side of patient compartment.
2. Position the mounting base 1 inch from the front edge and 1/2 inch from the top shelf as shown.
3. Using the charger base guide pins as a template, drill two 1/8" holes.
4. Squeeze two 1/4" beads of silicone on to the base bottom and install to the wall with the supplied self-tapping screws.
5. Insert the power cord and mount the charger onto the base. Stick the double sided tape to the transformer and attach to the wall behind the charger. Secure the excess cables with zip ties.

