

STRYKER BATTERY CHARGER INSTALLATION

September 26,
2017
Page 1 of 1

AMBULANCE TYPE: DEMERS 62800 - 62999

UNITS AFFECTED: ALL UNITS EQUIPPED WITH THE STRYKER POWERPRO
STRETCHER

ACTION: INSTALLATION OF THE STRYKER POWERPRO BATTERY CHARGER

LABOUR TIME: .3 HRS **BILL TO:** BCAS

SERVICE PROCEDURE:

1. Remove the storage bins to provide space to mount the Charger on the left of the lower shelf located in the cabinet above the suction unit.
2. Position the mounting base $\frac{1}{2}$ inch from the side wall and $\frac{3}{4}$ inch from the front door track.
3. Using the charger base guide pins as a template, drill two $\frac{3}{16}$ th holes through both the shelf and the lower cabinet frame.
4. Squeeze two $\frac{1}{4}$ " beads of silicone on to the base and install on the shelf with the supplied machine screws, lock washers and acorn nuts. **Tighten snug but do not over tighten as damage to the shelving unit will occur.**
5. Insert the power cord and mount the charger onto the base. Stick the double sided tape to the transformer and attach to the wall behind the charger. Secure the excess cables with zip ties.

